

PLEASE MAKE SURE THE UPPER CLEAR TAB IS RESTING **ONLY** ON THE 5TH METATARSAL HEAD.



✓ YES

MAKE SURE TAB IS DIRECTLY ON THE 5TH MET HEAD.



✓ YES

ALSO MAKE SURE THE HEEL TAB IS REMOVED PRIOR TO SCANNING.



✗ NO

5TH TAB IS TOO FAR IN AND RESTING ON 3RD AND 4TH MET HEADS.



✗ NO

5TH TAB IS TOO LOW AND RESTING BELOW 5TH MET HEAD.



✗ NO

5TH TAB IS TOO HIGH ON THE 5TH TOE.