

SmartCast®

RAISE AND TILT CHAIR AS MUCH AS REASONABLY POSSIBLE (rev. 6/20)

Capturing the back of the patient's heel is arguably the most difficult part of the scanning process. Raising and tilting the chair to elevate the patient's foot (**after the foot has been positioned/aligned**) allows a seated practitioner to perform the scanning process at eye level.

This makes it easier to capture a great scan, particularly the back of the patient's heel, while requiring less bending.

Scanning patients on a low table is *possible* but lessens the likelihood of a good scan and can be awkward for the practitioner.

